Inkontinenztherapie-Stimulator
Stimulator for Incontinence-Therapy

BioBravo

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Content

Please read the Operators Manual carefully
before using the device and pay attention
to all references !

	Page
I. Incontinence Therapy BioBravo	
Delivery & Accessories - Measurement & Weight - EMC	27
Introduction: What is BioBravo - Indication	28
Contraindication	29
Advice on Application of Electrodes	30
Technical Data - Safety Adjustments	31
Maintenance - Warranty	32
Setup Real Time Clock	33
II. Display Elements of the BioBravo	
Display and Operating Elements	34
Description of the Device	35
III. Application of the BioBravo	
Settings of The Preset Programs	36-37
Settings of the Individual Programs	38-39
IV. Incontinence-Therapy	
What is Incontinence	40
Different Types of Incontinence	41
Programs 01-09	42-46
V. Control of Therapy	
Memory	47
Reading of Memory	48
Literature	49
Waste Disposal - Copyright	50
Anatomy	51

Delivery & Accessories

Included in the delivery:

REF 010E-003: 1 device BioBravo, 2 lead wires, 4 self adhesive electrodes (40 x 40 mm), 4 self adhesive electrodes (40 x 80 mm), two 9 V batteries and 1 Operators Manual

REF 010E-038: 1 device BioBravo, 2 lead wires, 1 vaginal probe (IncoProbe V) 1 lubricant, two 9 V batteries and 1 Operators Manual

REF 010E-039: 1 device BioBravo, 2 lead wires, 1 rectal probe (IncoProbe R), 1 lubricant, two 9 V batteries and 1 Operators Manual

Accessories:

REF

011E-006: 1 unit self adhesive electrodes 40x40 (4 pieces)

011E-007: 1 unit self adhesive electrodes 40x80 (4 pieces)

011E-051: 1 unit self adhesive electrodes 32mm Ø (4 pieces)

011E-008: 1 vaginal probe "IncoProbe V"

011E-009: 1 rectal probe "IncoProbe R"

011F-012: 2 lead wires

011F-013: 2 pieces 9V batteries

011F-037: 1 tube electrode jelly for probes

011E-047: 1 operators manual BioBravo

011E-050: 1 storage box

BioBravo may only be used with original accessories.

Measurement & Weight

Measures of the device: H 14,1 cm / W 7 cm / D 4 cm

Weight of the device: 180 g (incl. battery)

Electromagnetic Compatibility

The device conforms to the electromagnetic compatibility requirements of European Standard EN 60601-1-2 "Medical Electrical Equipment, Part 1: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests"

Introduction – What is BioBravo ?

BioBravo is a neuromuscular stimulator, which has been developed according to latest findings by MTR+ in consultation with the medical profession and patients. This device is a modern 2-channel stimulator, which is often used with vaginal- or rectal-probes. Additionally this unit allows the use of 2 pairs of skin-electrodes (= 4 electrodes) to work simultaneously, too. **BioBravo** is a stimulation device, which allows both, high-efficient muscle stimulation for the pelvic muscles as well as being suitable for the calming of the hyper nervous bladder. This device can therefore be used for all forms of urinary and/or stool incontinence without any foreign assistance. Another program tries to make the activation of the peristalsis possible, so that this device can be used for remedying of obstruction, too.

Additionally the three freely configurable programs enable the specialist to compile specially tailored programs for his own individual purpose. This characteristic of the **BioBravo** offers therefore each physician or physiotherapist, but also the experienced patient, the possibility of a focused coordinated treatment for the respective state of health. As these programs, after storing, can be changed at any time, for example to be altered according to the progress of therapy, an almost unrestricted variation range is available with this device to the user.

After termination of the therapy, **BioBravo** can be re-used by other patients. Please clean it before with a mild disinfectant cleaner.

Indication

BioBravo was designed to aim at the following therapy of incontinence:

- Elimination of light to mid strength faecal incontinence (PRG 7, 8)
- Elimination of light to mid strength urinal incontinence
 - Urge-incontinence (PRG 1, 2)
 - Stress-incontinence (PRG 3, 4)
 - Mixed-incontinence (PRG 5, 6)
- Activation of peristalsis (PRG 9)

Contraindication

BioBravo may under no circumstances be used in the following cases:

- Patients with a pace maker as functional disturbances can occur
- Patients with myocard damages (damage to the cardiac muscle) or arrhythmia (inconsistent heart beat)
- When operating machinery with a potential of danger
- Patients with epilepsy
- When inflammation of the vagina, anus or urinary tract is present
- Patients with febrile illnesses or infectious diseases

Please pay unconditional attention to the following ground rules:

- Do not stimulate transcerebral (via the head)
- Do not stimulate via the Carotis-Sinus-Nerve (carotid artery)
- Do not stimulate in the area of the Sinus knot (heart)
- Do not stimulate on the Bulbous (eye)
- Do not stimulate in the Larynx-Pharynx-Area (larynx /pharynx)
- Do not stimulate in places with skin defects
- Do not position the electrodes directly on the spine

Take special care with the use of the electric stimulation:

- If after an operation the healing process can be interrupted through a muscle contraction
- After acute bleeding or bone fractures with a tendency to bleed
- In the case of the menstruation
- If in the case of damage to the perception nerves the normal sensation in the skin has been lost
- If at the same time the connection is made with a surgical high frequency apparatus, this could result in burn marks underneath the electrodes.
- If device operation occurs close to short waves or a microwave, discrepancies of the initial values of the stimulator could occur.
- The application of the electrodes nearby the thorax can increase the risk of ventricle cilia.
- In the case of pregnancy, stimulation should only be applied after consultation with the general practitioner.
- If in the case of treatment with the probe, pain or irritation appear in the area of the pelvis, or bleeding in-between menstruation, interrupt the therapy and consult your doctor.
- Should skin irritation occur whilst using the **BioBravo**, please interrupt the application and inform your doctor - to exclude an allergic reaction.
- Patients with metal implants may only stimulate with permission of an experienced doctor.

Please, keep this device stored out of the reach of children.

Advice on Application of Electrodes

Please empty the urinary bladder and clean the area to be treated.

Make sure that BioBravo is turned off (no display on the LCD display).

Now connect the single poled ends of the lead wires with the vaginal or rectal probe, respective the skin-electrodes.

Then fix the output plug(s) of the lead wires into channel A or B of BioBravo.

Furthermore make sure that all plugs fit completely into the electrode sockets as well as the device sockets.

If you are using a probe, make it gliding capable with water or a gel suitable for electrotherapy.

Please don't use any oil, ointment or cream, since all these have an isolating effect.

Now place the probe or the electrodes according to the instructions of your doctor or physiotherapists.

In the case of the skin gel loosing its adhesion, you can't get more than 6 mA - please replace the electrodes then. The same effect, however, also occurs when using a damaged cable. Therefore, if the normal treatment is not possible by using new electrodes, probably a cable is faulty.

If you should use a probe, please clean it after use carefully with a normal detergent and dry it completely.

Electrodes/probes which were damaged due to an improper handling may not be used any more.

Electrodes/probes with a damaged cable isolation (e.g. visible copper wire may not be used any more).

Electrodes/probes may not be changed (e.g. by means of scissors or other tools).

Do not pull the electrodes/probes by force on the electrode cable.

Apply the electrodes on clean and unwounded skin only.

In the case of strong bodily hair growth, the hair should be shortened with scissors.

Neither stretch the skin nor the electrodes when applying the electrodes.

To remove the electrodes from the skin, please lift the electrodes at the rim towards the middle.

Effective current densities of more than 2 mA/cm² require an increased attention of the user with regard to the intensity adjustment.

We recommend the use of our vaginal- (IncoProbe V) or rectal- (IncoProbe R) probes.

Technical Data

Channels:	Two isolated independent channels
Waveform:	Asymmetrical, rectangular, bi-phasic with zero DC current
Intensity:	0 - 90 mA
Frequency:	1 - 200 Hz
Width of impulse:	50µs - 450µs
Ramp up/down time:	0,1 seconds - 9,9 seconds
Time setting:	variable: 1 minute - 9 hours + 59 minutes
Electrical supply:	9 volt block battery
Resistor:	500 Ohm 1000 Ohm 2200 Ohm
U max:	48 V 80 V 120 V

Environmental conditions for storage, transportation & use: -10 to +50 degree centigrade, 0-90 % atmospheric humidity, air pressure of 700-1060 hPa

Explanation of Symbols

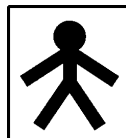
Attention!

Please consider
Operators Manual

0123
CE-Sign

Grade of protection BF

This device is quality controlled and fulfils the rules and regulations set by the German law for medical products (MPG) as well as the guidelines of the European Community 93/42/EWG for medical products. Therefore this device carries the CE-sign „CE 0123“.

You can find the serial number of this device in the battery compartment.

Manufacturer: MTR+ Vertriebs GmbH, Kamenzer Damm 78, 12249 Berlin

Safety Adjustments

1. The +buttons become inactive after one minute running time of a program to avoid an unintentional increase of intensity . This lock is switched off again, only by pressing the +button three-times repeatedly. Now an increasing of intensity is possible again for another minute.
2. If the electrical circuit should be interrupted during operation (> 6 mA), for example by losing an electrode or by a broken lead wire, the device stops stimulating immediately.
3. If any phase comes to its end the intensity falls down to 8 mA automatically and must be regulated by the user on a pleasant measure again.

Maintenance

BioBravo has been designed to be maintenance free. Nevertheless, please adhere to the following advice:

If needed please clean **BioBravo** and the probes with a mild detergent, water and a damp cloth. Do not expose **BioBravo** to extreme humidity and do not submerge under water.

You may clean the cables with a damp cloth. You may also from time to time treat the cables with talcum powder in order to prevent cracks and to further ongoing usage.

When changing the battery, please make sure to insert the new battery at least one hour after removal of the used battery. We recommend always storing a spare battery in the equipment bag.

The **BioBravo** may not be operated if the battery compartment is not closed. Before opening the battery compartment, make sure that there are no cables in the jacks at the head of the unit to eliminate any possibility of current flow.

Never connect the device directly to a battery charger or to any other mains powered equipment.

Please subject the device to technical safety control in regular intervals in accordance with the local regulations.

Warranty

MTR+ Vertriebs GmbH provides a 24 month warranty, commencing from the date of our delivery (invoice date), that in case of failure, the **BioBravo** will be serviced or replaced free of charge. All returns must first be authorised by MTR+ Vertriebs GmbH in advance.

Excluded from this warranty are failures of the **BioBravo** which result from excessive use, careless or improper handling as well as damage by force.

The warranty loses its lawful meaning if the producer MTR+ Vertriebs GmbH has not given its consent in writing prior to repairworks or servicing being carried out. Without this written consent the manufacturer will not return the device.

The manufacturer reserves the right of technical changes, which may serve to increase treatment success, the functionality of the device or its accessories.

Setup Real Time Clock

BioBravo has a “Real Time Clock”, that shows year, month, day, hours and minutes to allow the therapeutic memory the temporal assignment of detailed treatment data..

To reach the setup-mode keep the PRG-button pressed for ten seconds.

With the +/- button of channel A (left side) you can step forward to the following values

- hour (indicated by: H)
- minute (indicated by: MI)
- month (indicated by: MO)
- day (indicated by: dA)
- year (indicated by: Y)

The respective value is flashing and can be changed with the +/- buttons of channel B (right side).

Finally you save all your settings by pressing the PRG-button once again.

Before using memory via the lock button (see page 43), please set the clock.

Display and Operating Elements

„ON/OFF“: By pressing the button once, the device will either be switched on or off.

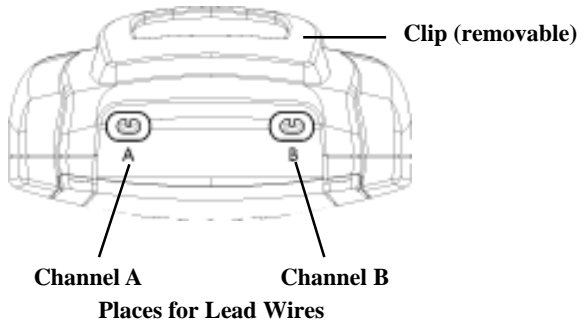
„+/- buttons“: This button regulates the intensity for the channels A and B. The device also has two independently controllable output channels A and B. When pressing the respective arrow buttons the intensity will either be increased or decreased in steps of 1 mA. 60 seconds after the last intensity regulation, a further increase may not be carried out due to an electronic barrier. But if the „+“ button is pressed three times, the barrier is released and the intensity can be altered again. A reduction of the intensity is possible at any time. Whilst configuring the programs 10 – 12 one switches between the programmable parameters by pressing the left „+/-“, button (channel A), and through pressing the right „+/-“, button (channel B) the desired value/data for respective parameter can be set. In this way, the application time, the stimulation, the pause, rising pulse and falling pulse, as well as synchronous or asynchronous stimulation can be selected.

„PRG“: With this button the preset programs 01–09 or the three free programmable programs 10–12 can be selected for individual use. Pressing the PRG-button once at any time during a running program will reduce the mA on both channels to zero and pause the clock display with the actual treatment time. The clock continues to show the time of further treatment but it is frozen with the “:” symbol no longer flashing. By prolonged pressure on the PRG-button (10 seconds) you’ll reach setup of the clock. In addition, this button stores the current adjustment in the programming mode and ends it.

„SET“: With the SET-button you can select the different program phases. With every pressing of this button, the phase in progress is terminated and the next program phase is started. This applies to either the running of the programs 04–06 and 08–09 and also to the programming and the sequence of programs 10–12. Programs with one single phase will be finished by pressing the SET-button. If you keep the SET-button pressed for 3 seconds whilst being in programs 10–12, the symbol of the clock is flashing and you will be able to start programming of your individual settings.

Description of the Device

Settings of the Preset Programs 01–09

Preparations:

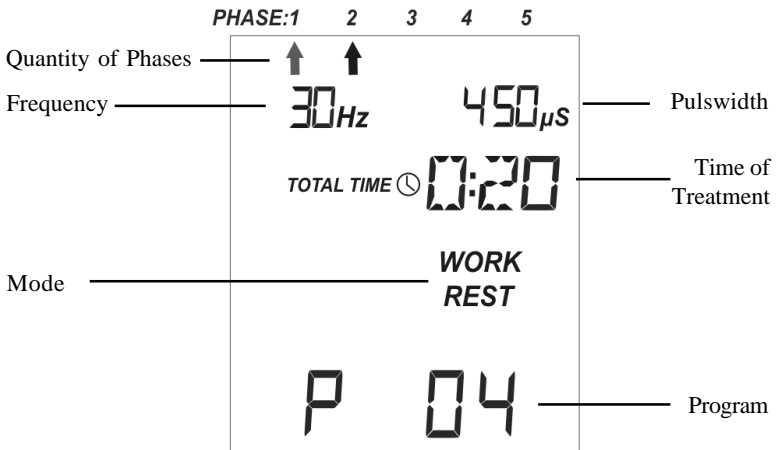
Make sure that **BioBravo** is switched off. Cleanse the area of the skin which is to be treated with clean water. Following that, make sure that all plugs are completely connected to the respective electrode sockets as well as device sockets. Place the vaginal-/rectal-probe or the adhesive electrodes onto the area that has been prior cleansed, according to the advices in pages 29-30 of this manual. The clip on the back of the unit allows to fasten the device at the clothes.

BioBravo is operated by an 9 volt block battery. Open the battery compartment at the back by pushing down the lid and put in the battery by hand. Please ensure the correct polarity in accordance with the markings. **BioBravo** is now operational. If the battery voltage is too low, the battery-symbol in the LCD-display will flash. Please replace the battery at this time. Empty batteries constitute special waste and therefore have to be disposed of at an appropriate collecting point. Never dispose of batteries via domestic or household waste.

Adjustment:


1. Press the button „On/Off“ once.

BioBravo is now switched on. The program used last, appears on the LCD-display. Parameters programmed for the first phase will be displayed as well as the quantity of all phases, the actual treatment mode and the total treatment time.

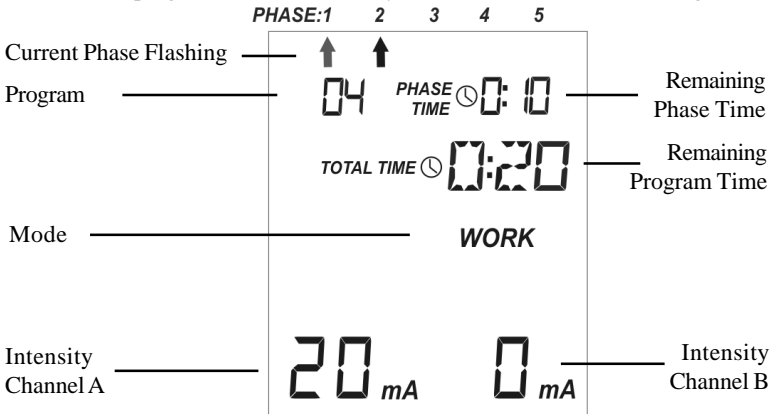- By pressing the program button „PRG“ you can choose from the programs 01 – 09 (pre-programmed).
- This is followed by pressing the buttons „+/-“ below the channel used.. The selected program runs automatically now and the LCD-screen changes.

The current phase will be indicated by an flashing arrow and in the upper right corner you can see the remaining phase time. In the bottom line the intensity used in each channel appears. If the button „+“ is pressed repeatedly the intensity of stimulation increases. If the button „-“, is pressed repeatedly the intensity of stimulation is reduced. For the setting of the intensity it is vital that one feels the electric current but that pain is never incurred.

Please take into account that for safety reasons the intensity falls back at every changing of phase on the value of 8 mA and must be regulated by you on the desired standard again.

Each phase end is indicated by 5 peeptones.

- If you wish to interrupt the program, then press the button „PRG“ once. The program will run again for the remaining program time when pressing the button „+“. In this case, the intensity must be adjusted newly, too.
- By pressing the „SET“ button you end the current phase and move to the next. Programs with one single phase will be finished this way.
- BioBravo** is switched off when pressing the button „On/Off“ or automatically four minutes after the program is finished.

Settings of the Individual Programs 10-12

1. Switch on **BioBravo** with “**On/Off**” button.
2. Choose the desired program (10-12) with “**PRG**” button.
3. Press “**SET**” button for at least three seconds. **BioBravo** changes to the programming mode and the upper watch, which indicates phase duration, flashes to show that this function can be altered.

In the following please use the left “+” button always then, if you want to store your choice and to change to the next step. The right “+/-” buttons are respectively used in order to alter the chosen and flashing parameter.

4. Select with the right “+/-” buttons the desired time for the first phase.
5. By pressing the left “+” button you will be able to store your choice and change to the next step. In the display now the mode flashes, which you adjust according to your wishes by pushing the right “+/-” buttons.
If you have chosen “**CONT**” or “**BURST**” you can determine frequency and after storage and change with the left + button respectively the pulse width by pushing the right “+/-” buttons.

Please note, that it is basically not possible to program more than 300 μ S if frequency is above 100 Hz.

If you have chosen “**WORK/REST**”- rhythm you can determine the following parameters one after the other by pushing the right “+/-” buttons.

- frequency (**Hz**)
- pulse width (**μ S**)
- work time (**sec**)
- rest time (**sec** = recovery time)
- increase / decrease ramp up time of the pulse (**RU**)
- increase / decrease ramp down time of the pulse (**Rd**)
- synchronous (**SY**) or alternating (**AL**) work of the channels A and B. If you have selected “**SY**” you can program **delay-time (dL)** of 0 sec. up to 5 sec. between the channels A and B.

If you have chosen **“MOD”** you can alter the following parameters one after the other.

- low limit of frequency (**Hz LO**)
- upper limit of frequency (**Hz HI**)
- low limit of pulse width (**µS LO**)
- upper limit of pulse width (**µS HI**)
- low limit of intensity (Adj) - the display in the right corner below shows values from 0.5 to 1.0. This means that the intensity of current can increase and decrease within the modulation time between 50% (=0.5) and 100% (=1.0) of the original adjusted mA-value. The upper limit of the intensity, that is 100%, is always the mA value adjusted by the user originally. By the fact that the intensity, even in a modulated program, can never increase above the 100%-value it is ensured, that there will be not more current, then the user has individually regulated himself during the whole stimulation process.
- modulation time (possible choice: 2 sec. up to 60 sec)

6. By pushing the **“SET”** button, you reach the second phase of the program. You can analogously program these to the first sequence now. This way you can make up to 5 phases of your own in each of the three configurable programs. If you should want to reduce a configured program of 5 phases to less phases, you simply put the time of the first unwanted program section to zero.

7. The programming is completed by pressing the **“PRG”** button and the program is stored.

Therapy for Incontinence

What is Incontinence ?

The term "incontinence" refers to the inability to deliberately withhold urine and/or bowel movements. The causes of incontinence are malformations, illness or injury.

Urinary incontinence is more common than one might initially suspect. It looks that the number of people affected by incontinence will increase steadily. By the year 2050 according to various estimates nearly 30 % of people could be affected by incontinence. About 80% of women between 20 and 75 years and 10% of men suffer from urinary incontinence. The faecal incontinence in both sexes is about 10%.

Incontinence is not only a medical problem, but also represents a significant psychological burden. Many affected individuals respond with depression and withdrawal from social life. Incontinence should never be tolerated as fatefully, since most patients can be helped with proper treatment. Forms of incontinence, upon which medicines have little effect are well therapeutible by electric stimulation very often.

The therapy by electric stimulation is not stressful, it is simple in its application and allows wide ranging use for treating bladder and discharge malfunction as well as faecal incontinence.

The flow of electricity that is generated by means of probes or adhesive electrodes, leads to a pelvic floor contraction, which the patients feel. They learn quickly to control the muscles of the pelvic floor itself again. The improvements can be measured by the appropriate use of a biofeedback device. Experience has shown that often you can achieve significant therapy progress within a short time, by using electric stimulation combined with biofeedback.

The aim of the electric stimulation is as follows:

- focused training of the pelvic muscles
- strengthening of the bladder closing muscle
- attenuates the highly active bladder muscle
- regaining the physiological sensation of the bladder
- strengthening of the sphincter muscle
- activation of peristalsis

Different Types of Incontinence

BioBravo can be applied with the following types of incontinence.

Stress incontinence means that the group of muscle that belongs to the closing mechanism of the bladder exit is not fully functionable. When pressure rises (e.g. through sneezing, coughing, laughing or physical exertion) the loss of urine will occur even without micturition. The cause is a weakening of the pelvic muscles which can result from either weakness of the connective tissue, birth or prostate surgery.

Stress incontinence is classified according to three severity levels:

- **Grade 1:** Involuntary loss of urine during violent increase of pressure in the abdominal area, eg by sneezing, coughing, laughing or heavy physical exercise.
- **Grade 2:** Involuntary loss of urine in moderate increase of pressure in the abdominal area, eg by walking or climbing stairs.
- **Grade 3:** Involuntary loss of urine with only a slight increase in pressure in the abdominal area even when lying.

Urge incontinence means that the closing mechanism of the bladder and urethra are mostly functioning. Often it comes to extrude movements of the bladder, which cannot be suppressed through mind power. One speaks of a hyperactive bladder muscle. Micturition is uncontrollable and the pressure of the bladder increases to the point, that the resistance of the healthy muscular closing system will be overcome. Possible causes are disorder of the superior nervous system.

A **mixed form of stress and urge incontinence** means that the cause lies with the weakness of the muscular closing system as well as a hyperactive bladder muscle. In most cases one of the two forms is especially strong.

Faecal incontinence means a disorder of the function of the sphincter anus externus. A controlled discharge of bowel contents is not possible anymore. The causes for the functioning disorder of the sphincter anus externus are manifold. There are various different levels:

- **Grade 1:** Uncontrolled disposal of winds / Light soiling of the laundry
- **Grade 2:** Uncontrolled discharge of fluid stool / Uncontrolled disposal of winds / Occasional uncontrolled stool
- **Grade 3:** Loss of stool and all the winds totally uncontrolled

Programs

Progr.: 01		Urge incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		Cont				
Frequency work	Hz	10				
Frequency rest	Hz					
Pulse duration	μ S	250				
Modulation time	secs					
Ramp up time	secs					
Ramp down time	secs					
Work time	secs					
Rest time	secs					
Alternating						
Synchronous						
Overall time	20 min					

Calming of the hyperactive bladder.

Progr.: 02		Urge incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	10				
Frequency rest	Hz					
Pulse duration	μ S	350				
Modulation time	secs					
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6				
Rest time	secs	15				
Alternating						
Synchronous						
Overall time	20 min					

Calming of the hyperactive bladder.

Programs

Progr.: 03		Stress incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	30				
Frequency rest	Hz					
Pulse duration	μ S	200				
Modulation time	secs					
Ramp up time	secs	0,8				
Ramp down time	secs	0,8				
Work time	secs	5				
Rest time	secs	8				
Alternating						
Synchronous						
Overall time	20 min					

Training the weakened pelvic muscle. This program can be used with faecal-incontinence as well.

Progr.: 04		Stress incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10			
Mode		W/R	W/R			
Frequency work	Hz	30	40			
Frequency rest	Hz					
Pulse duration	μ S	450	300			
Modulation time	secs					
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	6	6			
Rest time	secs	15	15			
Alternating						
Synchronous						
Overall time	20 min					

Training the weakened pelvic muscle. This program can be used with faecal-incontinence as well.

Programs

Progr.: 05		Mixed incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	3	10	5	4	3
Mode		W/R	W/R	W/R	W/R	W/R
Frequency work	Hz	3	10	20	30	40
Frequency rest	Hz					
Pulse duration	μ S	250	250	250	200	200
Modulation time	secs					
Ramp up time	secs	0,8	0,8	0,8	0,7	0,7
Ramp down time	secs	0,8	0,8	0,8	0,7	0,7
Work time	secs	4	4	4	4	4
Rest time	secs	4	4	4	6	6
Alternating						
Synchronous						
Overall time	25 min					

Treatment of mixed urethral incontinence. Both the overactive bladder and the weakened pelvic muscle are treated.

Progr.: 06		Mixed incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10	10		
Mode		W/R	Cont	W/R		
Frequency work	Hz	12	10	35		
Frequency rest	Hz					
Pulse duration	μ S	400	200	300		
Modulation time	secs					
Ramp up time	secs	1		1,5		
Ramp down time	secs	1		1		
Work time	secs	6		5		
Rest time	secs	15		12		
Alternating						
Synchronous						
Overall time	30 min					

Treatment of mixed urethral incontinence. Both the overactive bladder and the weakened pelvic muscle are treated.

Programs

Progr.: 07		Faecal incontinence 1				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	20				
Mode		W/R				
Frequency work	Hz	40				
Frequency rest	Hz					
Pulse duration	μ S	200				
Modulation time	secs					
Ramp up time	secs	1				
Ramp down time	secs	1				
Work time	secs	6				
Rest time	secs	15				
Alternating						
Synchronous						
Overall time	20 min					

Training the sphincter muscle. This program can be used with stress-incontinence as well.

Progr.: 08		Faecal incontinence 2				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	5	10	5		
Mode		W/R	W/R	W/R		
Frequency work	Hz	40	35	30		
Frequency rest	Hz					
Pulse duration	μ S	300	400	450		
Modulation time	secs					
Ramp up time	secs	1	1	1		
Ramp down time	secs	1	1	1		
Work time	secs	5	6	5		
Rest time	secs	12	12	12		
Alternating						
Synchronous						
Overall time	20 min					

Training the sphincter muscle. This program can be used with stress-incontinence as well.

Programs

Progr.: 09		Activation of peristalsis				
		Phase 1	Phase 2	Phase 3	Phase 4	Phase 5
Phase time	min	10	10			
Mode		W/R	W/R			
Frequency work	Hz	8	12			
Frequency rest	Hz					
Pulse duration	μ S	450	400			
Modulation time	secs					
Ramp up time	secs	1	1			
Ramp down time	secs	1	1			
Work time	secs	5	6			
Rest time	secs	10	12			
Overall time	20					

In this program 09 it is tried to activate the peristalsis. To reach this target one uses both channels of the BioBravo in connection with self-adhesive electrodes, which are fixed crosswise on the body in the intestines area.

Abbreviations:

- AdJ: Adjustment of intensity
AL: Alternate function of the channels A and B
BURST: The burst mode delivers 2 blocks of 9 impulses each per second
CONT: In the continuous mode a permanent unchanged current takes place.
dL: Delay between channels A and B
dA: Day
HI: Upper value (high)
Hz: Hertz is the measurement unit for the frequency of impulses per second.
LO: Lower value
mA: Milli-amp is the measurement unit for the electrical current intensity.
MOD: In the modulated mode the frequency and/or the pulse width and/or the intensity changes in a predefined rhythm automatically.
Rd: Ramp down
RU: Ramp up
secs: Seconds
SY: Synchronous function of the channels A and B
TI: Time
TM: Time of modulation-interval
W/R: Work/Rest mode
 μ S: Microsecond is the measurement unit for the length of every single impulse.

Control of Therapy

BioBravo has a memory store, which can be controlled by the physician. The therapist can read informations about the compliance of the patient on the display at any time he wants. This way it is possible to evaluate whether the device was used according to the instructions of the doctor (quantity, intensity, etc.)

BioBravo offers the chance to get a quick survey about the stored data of all treatment-sessions. The global statistics display the following averages:

- number of treatments per day
- number of days in use
- most used program
- total time of use
- intensity per channel

If more detailed information is required, **BioBravo** allows to have a closer look at each single session of the last 60 treatment-days. You can see:

- number and date of the treatment-day
- number of session at this day
- time of session-start
- number of program used in this session
- total-time of the session
- intensity used for each channel in this session

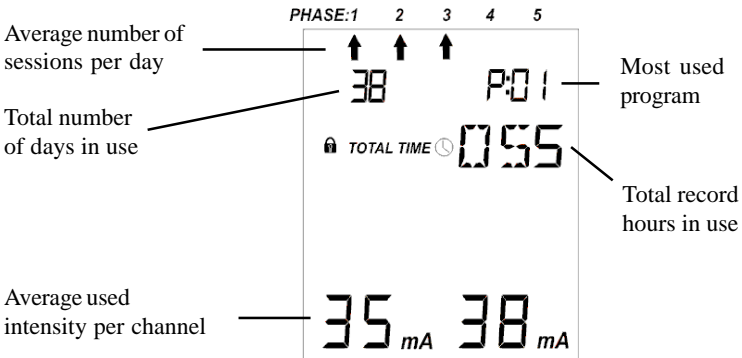
Before the different values of the treatment can be stored, it is necessary to activate the memory once. To achieve this aim, please make sure, that no cables are connected to the jacks in the head of the unit, switch on **BioBravo**, choose a program, open the battery-compartment and press the concealed "LOCK"-button with a thin stick (end of the lead wire) until you hear a **double** beep. On the LCD-screen appears the symbol of a padlock and the letters "P" or "T". By pressing any +/- button you can choose, whether you want to activate the memory in Time-mode (T) or in Program-mode (P). The Time-mode allows the use of all programs, while the selection of „P“ only the previously selected program makes available to the patient. To store, press the PRG button and the device is ready for use.

Reading of Memory

If you want to read the memory, switch on the device and press the LOCK-button in the battery compartment until you hear a **single** beep. The global statistics can be viewed on screen now.

Global Statistics:

To get more detailed information press the + button of the left channel (A). You will see the data of the first session of the first day. The next sessions of this day are displayed - one after the other - by pressing the +/- button of the right channel (B).

The next day can be obtained by pressing the + button of channel A again.

If you require information about time and date of one session please press SET. Then you can use any +/- button to switch from date to time and reverse. With SET you come back to the data of the last viewed session.

To stop the readout, press the PRG button for 3 seconds. For a new therapy control the device should now be re-activated (see page 47).

The data of the memory keep stored after reading, so that therapy-control is possible anytime. A maximum of 60 days of data can be stored with 5 sessions per day. When 60 days of data are reached, consecutive days will overwrite the earliest days. The day number will continue increasing (61, 62, ...) and be displayed when the unit is unlocked until the **BioBravo** is reset.

The statistics can be deleted completely. After pressing the LOCK-button you have to press SET, A+, A-, B+, and B-. This way all individual settings in programs 10-12 will be canceled as well. The **BioBravo** is set to it's default state and statistics are deleted.

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In accordance with European regulations [1], used electrical and electronic equipment may not be given to the unsorted waste anymore. The symbol of the litter bin on wheels indicates the necessity of separate collection.

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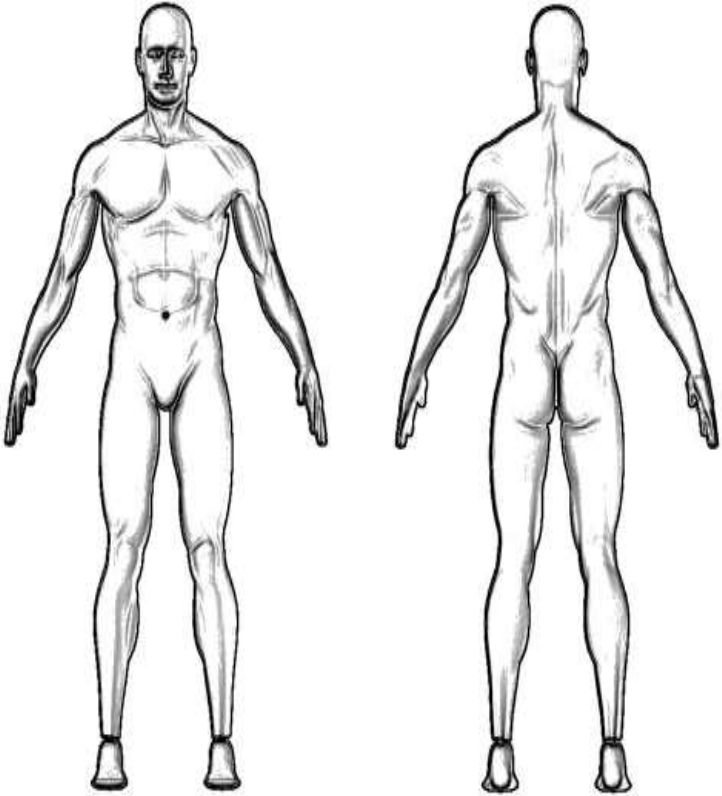
Reserved for changes.

Printing or copy of this manual - also in an abridged version - is forbidden and can only be allowed with a written approval of “MTR+ Vertriebs GmbH”.

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